


Get Started

DA1432 DECT indePendant™

Emergency Caller with Assistive Functions


DA1432 Overview

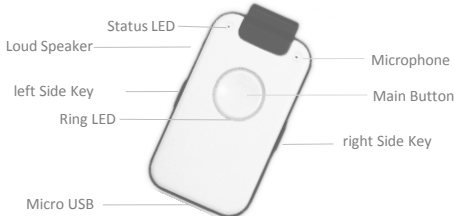

DA1432 is your everyday assistant with central call button
 If you need help, press Main Button and you will be connected with neighbours or friends.
 Fall Detection or Smoke Alarm automatically trigger help call
 All activities are conducted by the language assistant.
 Phone book (10 entries) with list of names is read

SET-into-OPERATION


step 1: Charge Device

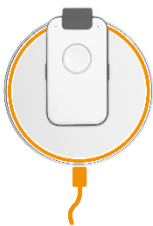

1. place Charging Disc on a flat, non-slip surface
2. connect the Charging Disc mains via supplied USB cable
3. place Device for approx. 30 min on Charging Disc


Note: Device will get a little warm; this is normal

step 2: Register Device to Base Station

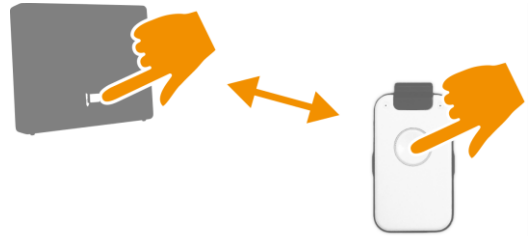

1. place Device close to DECT base station
2. activate Registration mode of the base station
3. press Main Button of Device to start the Registration
4. pay attention to announcements of Voice Assistant

step 3: check telephone connection

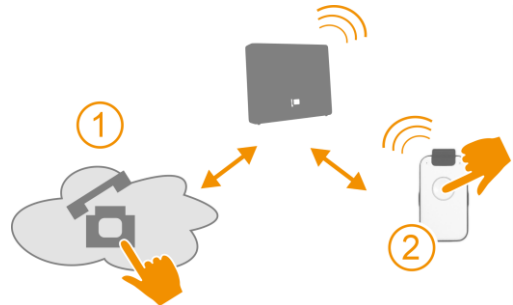

1. call Device with your mobile phone
2. if Device rings, answer the call

Note: The calling number is stored as the first number to call for help, fall detection and smoke alarm call.

Change or supplement this number with the PC APP.

Device is ready to use

Congratulations

now You can

- receive phone calls
- call phone number (see family & friends)
- execute a Help Call

following features are enabled:

- Fall Detection with automatic help call
- Smoke alarm call


You can configure via PC APP:

- all contacts for emergency (help, fall, smoke alarm)
- more Family & Friends contacts
- Time and Reminder Messages

CONFIGURATION



start PC APP

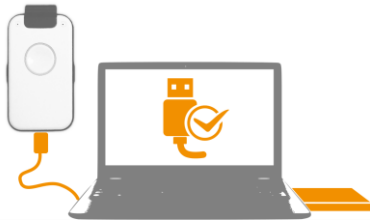

1. connect enclosed USB stick to PC
2. select drive "indePendant,, with WIN Explorer
3. start program "SETUP" (device driver)
4. connect Device via USB cable to PC
5. start program "PC_APP"
6. follow instructions of PC APP which are very simple

6

Tips

Charging : Make sure that Device is correctly placed on Charging Disc. LED indicates battery level status during charging. Note that Device cannot be registered, if battery is not sufficiently charged.

Registration: Please keep in mind that DECT base stations exit Registration mode automatically after 5 min.

Use PC APP to register the device, if PIN of the base station is not '0000'.

Trigger Help Call: Select and test "Ringing Time" in order to ensure, that the called party has sufficient time to answer the call. The ringing time should not too long in order to not delay the call of the subsequent numbers . Better choose more repetitions in order to find a contact quickly and safely.

9

Test Device functions



trigger Help Call



1. press Main Button > 2 sec
2. Help Call Sequence is starting
3. cancel Call Sequence by pressing Main Button again

7

LEDs

Color	Display	Status LED	Ring LED
RED	flashes	DECT or Battery out of range / order	Help Call is setup
	constant	charge ca. 60 min	Help Call is active
YELLOW	flashes	DECT or Battery sufficient	- DECT Registration - Fall Detection active
	constant	charge ca. 30 min	connected with PC APP
GREEN	flashes	DECT or Battery OK	Family&Friends Call is setup
	constant	Battery is fully charged	Family&Friends Call is active

10



Call Family & Friends



1. press side key to select wanted number (name)
2. press Main Button to setup call
3. cancel call by pressing Main Button again

8

Product Information

Title DECT indePendant™
Type DA1432
Ident Number 1432.1000.01

You will find a comprehensive Manual in the Internet and on enclosed USB stick

DOSCH&AMAND PRODUCTS GmbH

www.da-products.com

Neumarkterstr. 18
D-81673 Munich
Germany

sales@da-products.com

11